GOODGUT-TEST™
Manual de usuario/User
manual
V02.04.00

Contenido / Contents

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The purpose of this manual is to indicate the operating instructions of the product GoodGut-Test™ V: 02.04.00.

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This manual is exclusively for the use of product GoodGut-Test™ V: 02.04.00 by product users.

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The product GoodGut-Test™ V: 02.04.00 can only be used as a complement to the kits developed at GoodGut: RAID-CRC, RAID-Dx, and TestUrGut.

1.4. Referencias / References

- IMDRF/GRRP WG/N52 FINAL:2019 – Principles of Labelling for Medical Devices and IVD Medical Devices
- IVDR Regulation (EU) 2017/746

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3. Roles y Responsabilidades / Roles and Responsibilities

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The roles and responsibilities involved in the process are the following:

Hospital administrator

- *Manage the team of physicians and technicians from their hospital/centre to access the platform.*

Technician

- *Operation of the medical device following the instructions of this manual and the instructions of RAID-CRC, RAID-Dx, and TestUrGut kits.*
- *Issuing of the final report.*
- *Recording of incidences with device use.*

Doctor

- *Results interpretation of the final diagnostic.*

GoodGut personnel

- *Manage the access and permissions of the different centres/hospitals to the platform for the analysis of the products developed in GoodGut.*
- *Manage centres and permissions of the different users of the platform for the analysis of the products developed in GoodGut.*
- *Manage the registration of compatible thermal cyclers for the analysis of products developed at GoodGut.*
- *Manage the registration of qPCR kit lots for the analysis of products developed at GoodGut.*
- *Investigation of claims, incidences, and non-conformities detected.*
- *Open platform design changes when appropriate.*
- *Update the functionality/usability of the platform when appropriate.*

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Manufacturer: *GoodGut S.L.U.*

UDI: *(01)08437023437063 (11)202404(8012)020400*

Product classification: *complement of the in vitro medical devices RAID-CRC, RAID-Dx, and TestUrGut class C, B, and B respectively according to Regulation 2017/746.*

Intended purpose:

The GoodGut-Test™ platform complements three in vitro diagnostic medical devices developed by GoodGut: RAID-CRC, RAID-Dx, and TestUrGut. These tests base their diagnoses on the detection and quantification of specific microorganisms in stool samples. Detection and quantification of markers are done by qPCR.

The GoodGut-Test™ platform must be used to obtain the diagnosis. GoodGut-Test™ is a web application where sample data and the raw data of the qPCR must be introduced. Data associated with samples can be introduced manually one by one or data upload can be done massively (Sample menu). The needed sample information associated with the sample guarantees the correct traceability and that the sample accomplishes the requirements for reliable results.

The raw data obtained from the software associated with the thermocycler must be introduced into an Excel template provided by the platform for each processed sample (Analysis menu). The Excel templates and the number of templates to upload differ according to the analysed test. To obtain the correct templates, they must be selected from the GoodGut-Test™ platform. The excel files must contain the sample ID, which guarantees the traceability of the sample, together with the raw data obtained from the analysed markers of one of the qPCR multiplex of each analysis. The templates can be uploaded individually (single analysis) or massively (introduction of multiple analyses of the same product performed with the same kit at once). Once the templates are uploaded with the information, the platform applies the corresponding algorithm and generates a patient diagnosis report. The algorithms applied to obtain a diagnostic are found in the Thermocyclers menu. In this menu, the algorithm information is introduced for each product according to the selected thermocycler.

The GoodGut-Test™ platform includes a menu for the management of the qPCR lots and kits. The Lots menu allows the administrators to upload new lots of qPCR kits for the in vitro medical devices RAID-CRC, RAID-Dx, and TestUrGut into the system, including the type of analysis and the input route in the case of qPCR kits for RAID-CRC analysis, the lot number, the expiration date, the number of kits per lot, and the number of reactions per kit. The accepted range of values for the positive controls included in each lot is also specified in this menu, specifically in the thermocycler configuration section. The Kits menu allows the technicians to upload the kits currently in use in their laboratory to track the number of reactions performed and establish a relationship with the analyses uploaded in the system.

For the RAID-CRC and RAID-Dx tests the report obtained from the platform presents a qualitative diagnosis, positive or negative (incompatible) of the specific condition being analysed, together with quantitative data of the analysed microbial markers. For TestUrGut the report shows qualitative data (traffic light) and quantitative information of the analysed microbial markers. A specialist is responsible for giving the diagnosis to the patient.

The medical indication as well as the target population is associated with the tests.

Referring to RAID-CRC, the medical indication is for patients who need to be screened for advanced colorectal neoplasia, including precancerous lesions (advanced adenomas) and colorectal cancer, by detecting specific bacterial markers in faecal samples. There are two variants of this test according to their target population, individuals with an intermediate or high

risk of suffering from CRC, defined as follows: 1) Asymptomatic individuals aged 50 years or older, and 2) Individuals with symptoms compatible with CRC such as rectal bleeding, change in bowel habit, anaemia, unexplained weight loss, diarrhoea, iron deficiency anaemia, and/or abdominal mass.

Regarding RAID-Dx, the medical indication is for patients who need to be diagnosed with Irritable Bowel Syndrome and its differential diagnosis with Inflammatory Bowel Diseases, through qPCR detection of specific microbial markers in stool samples. The target population is individuals who present abdominal pain, stool alteration, and/or meet the Rome IV criteria.

About TestUrGut, the medical indication is for patients who want to be diagnosed with bacterial dysbiosis by detecting bacterial biomarkers in faecal samples. The target population is those subjects who, by prescription of a specialist or on their own, request an analysis of the state of the intestinal microbiota from GoodGut.

Intended user: *The intended user profile is a lab technician specialized in molecular biology. The intended user is responsible for analysing RAID-CRC, RAID-Dx, and TestUrGut kits in a clinical lab. Once the samples are analysed, the user must use the GoodGut-Test™ platform to obtain the final diagnosis.*

Principles of operation: *The different kits developed by GoodGut are in vitro diagnostic medical devices based on the analysis of specific microbial signatures for each product. The microbial signatures were developed by analysing stool samples from patients who participated in performance studies.*

The biomarkers analysed are different according to the pathology of interest. Once the samples were analysed, with the corresponding biomarkers, algorithms capable of differentiating patients with the target pathology were developed. The algorithms were validated with patients recruited in new performance studies. The GoodGut-Test™ platform applies these predetermined algorithms.

Related materials: *The GoodGut-Test™ product must only be used in conjunction with GoodGut developed in vitro diagnostic medical devices: RAID-CRC, RAID-Dx, and TestUrGut. Since GoodGut-Test® is a web application, no other associated material is required.*

Performance specifications: *The GoodGut-Test™ platform is a complement to the in vitro diagnostic medical devices developed by GoodGut. The platform only applies the predesigned algorithms of the associated products. Its operation has been verified and validated with 100% functionality. The analytical and clinical performance of the different products can be found in the corresponding technical document.*

Hardware specifications: *To use the GoodGut-Test™ platform, you only need a computer with internet access.*

Use conditions: *The GoodGut-Test™ intended use environment is the clinical laboratory where samples are analysed using the RAID-CRC, RAID-Dx, and TestUrGut kits. The platform is intended to be used indoors since it is a web application. No specific conditions of use are required.*

Warnings, precautions, contra-indications and other limitations of use: *The product is a web application. No hazards or hazardous situations are related to the usability of the platform GoodGut-Test™. The risk assessment carried out for the platform is reported in the document platform risk management file.*

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The GoodGut-Test™ platform is a complement to the in vitro diagnostic medical devices developed at GoodGut. Therefore, for the correct operation of the platform, the samples must be analysed using the kits: RAID-CRC, RAID-Dx, and TestUrGut following their instructions for use.

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The instructions for use detailed below are for obtaining a final diagnosis after analysis of the samples with the RAID-CRC, RAID-Dx, and TestUrGut in vitro diagnostic medical devices.

To use the platform, the user will receive an email where the GoodGut team welcomes the user and invites him/her to register through a link. The user must use the link to establish their password.

- *The password must contain:*
 - *A minimum of 8 characters*
 - *At least one capital letter*
 - *At least one number*
 - *At least one special character*

After registering, the user can access the screen for the System Login: <http://goodgut-test.eu/auth/login>.

- *You must enter the existing email and password in the system and then press the "Sign in" button. In the eye option, the user can see the password entered.*
- *The system allows the user to enter the password up to 5 times incorrectly. If the user is not able to log in correctly after 5 attempts, the system will ask to retrieve the password. The password can be recovered when a user does not remember it, introducing the email used in the system to give access to that option.*
- *The system will request a password change every 6 months.*

- *For security purposes, the system will not allow any browser to save the user password.*
- *The user sees the main menu with the shortcuts to each of the analyses, depending on the permissions granted to their centre.*
- *The user may close his/her session from the system at any time by displaying the user menu and pressing the "Log out" option. The system asks if you want to exit the system. If the user presses Yes, the system takes the user to the Login screen of the system. If the user presses No, the system keeps the user on the current screen.*

The user will be able to change the system's default language at any time it is required, using the option located in the top right corner of the screen. The available languages are Spanish, English, and Italian.

The user can notify the system administrators if they require any technical assistance from the platform. The option will always be visible at the bottom left of the screen. When activated, the system will invoke the messaging client (as configured on the device) to allow the user to send an email to support@goodgut.eu.

Hospital Administrator User:

- *New user creation*
 - *When entering the system with the Hospital Administrator Role, the system displays the Users menu option.*
 - *The user sees a list of all the users assigned to its centre. If there is still no associated user, a message will appear detailing that there is no user.*
 - *To create a new user, the user must press the "New User" button. By pressing the button, the system opens a loading screen, which has the following fields:*
 - *Name: Text field, required.*
 - *Last Name: Text field, required.*
 - *E-mail: Text field, required.*
 - *Phone: Number field, not required.*
 - *Role: Combo Technician/doctor, required.*

- *Service: Combo Gastroenterologist/family medicine/oncology, not required.*
 - *Hospital: Combo depending on the hospitals assigned to the hospital administrator, is required.*
 - *Country: Combo with options, not required.*
 - *The "Create User" button will be enabled once the required fields are completed. There is a "back" icon, which when pressed cancels the operation and returns to the user's list.*
 - *When pressing the "Create User" button, the system processes the creation of the user. If it is processed correctly, the system displays the following message: "The user has been added successfully". The system will save the created user and list it with the others.*
 - *If the user is processed incorrectly, the system displays the following message: "The user has not been created."*
- *User search*
- *The system displays a collapsible option for searching by filters. Below the collapsible filter is the results grid with the list of users. When displaying this option the system enables the following filters for the search:*
 - *Name*
 - *User*
 - *Role: Combo administrator/hospital administrator/technician/doctor*
 - *Service: Combo gastroenterologist/family medicine/oncology*
 - *State: Combo disabled/enabled/pending*
 - *Hospital: combo, this option depends on the number of centres sociated to the hospital administrator.*
 - *The "Reset Filters" button is available to eliminate the search data selected in the filters.*

- The "Apply" button applies the data selected in the filters. These will be reflected in the results list. The results grid presents the following fields:
 - Registration date: user registration date to the system
 - Name
 - Last Name
 - User's role
 - Service
 - Country
 - State

Once with the user's list, the hospital administrator has 5 different functions::

- The user has the option to edit the user by pressing the icon . The registration form opens where it is possible to modify the information.
- The user has the option to delete the user by pressing the icon: 
- The user has the option to enable or disable a user by pressing the icon:  for enableing and  for disableing. The system disables a user who has not logged in for a period of 6 months (180 days). The hospital administrator can enable it again.
- The user has the option of sending the invitation email for the registration again by pressing the icon: 
- The user also has the option to view more information about the user by pressing the icon: 

- Samples

- The user has access to the sample menu display that can be selected from the side menu.
- The user sees a list of all the samples of the selected product that have been carried out in their associated centre. If there is still no sample associated with it, a message will appear detailing to the user that there is no analysis.

- *Upon entering the Sample module, the system enables a collapsible search option. When displaying this option the system enables the following filters for the search:*
 - *Hospitals: combo, this option depends on the hospitals where the hospital administrator is associated.*
 - *Analysis type: combo, this option depends on the products active for the associated hospital.*
 - *Name: alphanumeric field*
 - *Last Name: alphanumeric field*
 - *Clinical history: alphanumeric field*
 - *Date of entry into the system*
 - *Date of deposition*
 - *Well name: alphanumeric field*
 - *Status: combo complete/expired/finished/finished expired/next to expire/not processable/pending/removed.*
- *The "Reset Filters" button is available to eliminate the search data selected in the filters.*
- *The "Apply" button applies the data selected in the filters. These will be reflected in the results list. The results grid presents the following fields:*
 - *Date of entry into the system*
 - *Analysis type: analysis associated to the sample*
 - *Clinical history*
 - *Well name: sample ID*
 - *Date of deposition*
 - *Status: status in which is the sample*
- *A sample is loaded so that when an analysis is run, it takes the data corresponding to it. For this process, the system will mark it with a different colour depending on its status.*
 - *Completed ■: Sample that was uploaded with all data pending the processing of the analysis.*
 - *Pending ■: Sample that was uploaded but there is missing data (FIT).*

- Deleted ■: Sample that was deleted without the analysis being performed by the user. The administrator can delete it either way.
- Due to expire □: Complete sample in the system for more than 8 days without the analysis being performed.
- Expired ■: Complete sample in the system for more than 10 days without the analysis being performed.
- Completed Expired ■: Completed sample in the system for more than 10 days and the analysis has been performed.
- Completed ■: Sample that was processed by analysis before it had been in the system for 10 days.
- Not Processable ■: Sample that cannot be processed because the patient is taking antibiotics or is pregnant.

Once the sample is found, the user has 1 function:

- The user has the option to view more details associated with the sample. Just press the icon: 

- Product analysis

- To view the reports of the different products, the user must select the type of analysis in the main menu using the RAID-CRC, RAID-Dx, and TestUrGut icons. It can also be selected from the side menu.
- The user sees a list of all the analyses of the selected products that have been carried out in their associated centre. If there is still no analysis associated with it, a message will appear detailing to the user that there is no analysis.
- Upon entering the Analysis module, the system enables a collapsible search option. When displaying this option the system enables the following filters for the search:
 - Date range: a calendar is enabled to select from and until the analysis
 - Sample code: alphanumeric field
 - Clinical history: alphanumeric field

- *Results: combo Positive/Negative regarding RAID-CRC and Positive/Not compatible in the case of RAID-Dx.*
- *Hospitals: combo, this option depends on the hospitals where the hospital administrator is associated.*
- *Technician: combo, this option depends on the number of technicians in the selected hospital.*
- *Route of entry: combo Screening/Symptomatic in the case of RAID-CRC.*
- *Status: combo, this option refers to the state of the analysis associated to the sample.*
- *Kit series number: alphanumeric field*
- *Lot number: alphanumeric field.*
- *The "Reset Filters" button is available to eliminate the search data selected in the filters.*
- *The "Apply" button applies the data selected in the filters. These will be reflected in the results list. The results grid presents the following fields:*
 - *Date and time: date of analysis entrance to the system*
 - *Sample code*
 - *Clinical history*
 - *Route of entry: refers to the Screening / Symptomatic analysis type in the case of RAID-CRC.*
 - *Results: Positive/Negative in the case of RAID-CRC and Positive/Not Compatible in the case of RAID-Dx.*
 - *Status: Status of the sample.*

Once the analysis is found, the user has 4 different functions:

- *The user has the option of placing observations by pressing the icon:  . A pop-up opens with a text field destined to enter, delete, or modify an observation, it will be related to the studies that were analysed at the same time.*

- The user has the option of writing a comment by pressing the icon:  A pop-up opens with a text field where you can enter the comments that will be issued in the report. This field accepts up to 1000 characters.
- The user has the option of downloading the report by pressing the icon:  A pdf file with the results report is downloaded.
- The user has the option of downloading the excel file that was entered into the system to analyse the study by pressing the icon: 

Technical User:

- New Sample creation

- When entering the system as a technical user, the system will show the menu option "Samples".
- The user can view a list of all the created samples in the same center. If there is no associated sample, a message will appear with the message: There is no sample.
- To create a new sample, the user must press the "New Sample" button. By pressing the button, the system opens a loading screen, which has the following fields:
 - Hospitals: combo, this option depends on the hospitals where the user is associated, mandatory.
 - Analysis type: combo, this option depends on the products active for the associated hospital, mandatory.
 - Route of entry: combo, with two options Symptomatic/Screen, only applicable when RAID-CRC analysis is chosen, mandatory.
 - Well name: alphanumeric field, mandatory.
 - Name: text field, mandatory.
 - Last name: text field, mandatory.
 - Clinical history: alphanumeric field, mandatory
 - Date of deposition: Calendar type field for single date selection, mandatory.
 - Antibiotics last month: single selection field Yes/No, mandatory-

- *Pregnancy: single selection field Yes/No, mandatory.*
 - *Aged 18 or over: single selection Yes/No, mandatory.*
 - *FIT ($\mu\text{g/g}$): only applicable when RAID-CRC is chosen.*
 - *Upper limit of detection: control enables or disables the FIT field ($\mu\text{g/g}$)*
 - *Well name: alphanumeric field*
 - *Status: combo complete/expired/finished/finished expired/next to expire/not processable/pending/removed.*
- *The "Create Sample" button will be enabled once the mandatory fields are completed. The "back" icon is available, when pressed cancels the operation and returns to the list of users.*
 - *By pressing the "Create Sample" button, the system processes the creation of the sample, when processed correctly, the system displays the following message: "The sample has been added successfully". The system will save the created sample showing it in the results grid with the others.*
 - *If the sample is processed incorrectly, the system displays the following message: "The sample has not been created".*
 - *There is the "+ Choose File" button which allows the user to load more than 1 sample at a time by selecting the file from their computer or directory. For a bulk sample upload, it is necessary to use a template so that the system can read the file correctly.*
 - *The "Download Template" button allows you to download a file model for bulk loading of samples.*
 - *This option disables manual user upload. Once the file is uploaded, it contemplates the same business rules as when manually entering a sample.*

Upon introducing a RAID-CRC sample regardless of route of entry, whether Symptomatic or Screen, if the FIT value (ng/mL) is below the pre-established cutoff point for analysis, an analysis with its corresponding results report will be automatically generated. The report will delineate the obtained FIT value (ng/mL)

resulting in a negative diagnosis. The values of Cq and the number of genomic copies/ μ L will be specified as "Not Relevant" (NR).

- Sample search

- *Upon entering the Sample module, the system enables a collapsible search option. When displaying this option the system enables the following filters for the search:*
 - *Hospitals: combo, this option depends on the hospitals where the hospital administrator is associated.*
 - *Analysis type: combo, this option depends on the products active for the associated hospital.*
 - *Name: alphanumeric field*
 - *Last Name: alphanumeric field*
 - *Clinical history: alphanumeric field*
 - *Date of entry into the system*
 - *Date of deposition*
 - *Well name: alphanumeric field*
 - *Status: combo complete/expired/finished/finished expired/next to expire/not processable/pending/removed.*
- *The "Reset Filters" button is available to eliminate the search data selected in the filters.*
- *The "Apply" button applies the data selected in the filters. These will be reflected in the results list. The results grid presents the following fields:*
 - *Date of entry into the system*
 - *Analysis type: analysis associated with the sample*
 - *Clinical history*
 - *Well name: sample ID*
 - *Date of deposition*
 - *Status: status in which is the sample*

- *A sample is loaded so that when an analysis is run, it takes the data corresponding to it. For this process, the system will mark it with a different colour depending on its status.*
 - *Completed : Sample that was uploaded with all data pending the processing of the analysis.*
 - *Pending : Sample that was uploaded but there is missing data (FIT).*
 - *Deleted : Sample that was deleted without the analysis being performed by the user. The administrator can delete it either way.*
 - *Due to expire : Complete sample in the system for more than 8 days without the analysis being performed.*
 - *Expired : Complete sample in the system for more than 10 days without the analysis being performed.*
 - *Completed Expired : Completed sample in the system for more than 10 days and the analysis has been performed.*
 - *Completed : Sample that was processed by analysis before it had been in the system for 10 days.*
 - *Not Processable : Sample that cannot be processed because the patient is taking antibiotics or is pregnant.*

Once the sample is found, the user has 2 different functions:

- *The user has the option to view more details associated with the sample. Just press the icon: *
- *The user has the option to edit the sample by pressing the icon . The registration form displays where it is possible to modify the information.*

- Creation of kits:

- *Upon logging into the system with a technical user, the system will display the "Kits" menu option.*

- *The user can view a list of all qPCR Kits that have been created at their center. If there are no associated kits, a message will appear indicating that there are no kits.*
 - *To create a new kit, the user must press the "New kit" button. Upon pressing the button, the system opens a loading screen, which includes the following fields:*
 - *Hospital: dropdown with options of the hospitals associated with the user, mandatory.*
 - *Analysis Type: dropdown with options of all analyses enabled for the selected hospital, mandatory.*
 - *Input Route: dropdown with Symptomatic/Screen options, only if the analysis type is RAID-CRC, mandatory.*
 - *Lot Number: dropdown with options of available batches at the center for the specified product.*
 - *Serial Number: the user must enter the serial number specified on the kit they want to input into the system.*
 - *The "Create Kit" button will be enabled once all mandatory fields are completed. There is an "back" icon, which when pressed cancels the operation and returns to the kits list.*
 - *Upon pressing the "Create Kit" button, the system processes the creation of the Kit and if processed correctly, the system displays the following message: "The kit has been successfully added." The system saves the created kit and displays it in the results grid with the others.*
 - *If the kit is processed incorrectly, the system displays the following message: "The kit has not been created".*
- *Kit search*
- *Upon entering the kit's module, the system enables a collapsible search option. When expanding this option, the system enables the following filters for the search:*

- *Hospitals: dropdown with the centres associated with this user.*
- *Analysis Type: dropdown with the active products in this centre.*
- *Batch Number: alphanumeric field.*
- *Serial Number: alphanumeric field.*
- *Date Range.*
- *Expiration Date.*
- *The "Reset Filters" button is available to clear the selected search data in the filters*
- *The "Apply" button applies the selected data in the filters. These will be reflected in the results list. The results grid includes the following fields:*
 - *Date and time of entry into the system: date of entry of the kit into the system*
 - *Hospital*
 - *Lot*
 - *Analysis Type: analysis associated with the kit*
 - *Input Route*
 - *Serial Number*
 - *Expiration date*
 - *Pending Reactions*
 - *Executed Reactions*
 - *Status: enabled or disabled kit icon*

Once any kit is found, the user has 4 functions:

- *Once the kit has been created, it can be enabled or disabled by pressing the icons:  and , respectively. The system will indicate with a padlock if the kit is enabled () or disabled (). Only one kit can be enabled per product at the same time. If an attempt is made to enable a kit when another kit is already active, the system will deactivate the current kit of the same analysis.*
- *A kit can be edited by pressing the icon: . Only kits that have not yet been used in any analysis can be edited.*
- *The details of a kit can be viewed by pressing the icon: .*

- *The executed and pending reactions of the kit can be viewed in the grid or by pressing the icon: .*

- *For the RAID-CRC analyses:*
 - *The RAID-CRC icon is selected from the main menu or it can be also selected from the side menu.*
 - *The user will see a list of all the RAID-CRC analyses that he has carried out. If this is the first time you log in and no analyses have been made, a message will appear stating that there is no RAID-CRC analysis.*
 - *To carry out a new analysis, the user must ensure it exists an habilitated kit. To proceed with the analysis, the user must press the "New Analysis" button. By pressing the button, the system opens a loading screen, which has the following fields:*
 - *Hospital: Combo to select the hospital or centre in which the analysis is carried out. It is a required field.*
 - *Thermocycler: Combo to select the thermal cycler with which the samples have been analysed. It is a required field.*
 - *Route of entry: Combo, two options: Screening or Symptomatic. The user must choose this field depending on the kit used to analyse the sample: RAID-CRC Screen or RAID-CRC Symptomatic. It is a required field.*
 - *Selecting a route of entry enables a field to attach the corresponding files. It only supports files in "*.xlsx or *.xls" format. It is a required field.*
 - *Screen route needs 3 different files (SCR_A, SCR_B, and SCR_C)*
 - *Symptomatic route needs 1 file (SYM)*
 - *The system also presents a "Download template" option, in which the user can download a correct template model of each corresponding file so that they can be completed without errors.*

- Screen route has 3 different templates (SCR_A, SCR_B, and SCR_C)
- Symptomatic route has 1 template (SYM)

Each template (SCR_A, SCR_B, SCR_C, and SYM) has a single tab that must be filled in following the instructions for use of the RAID-CRC Screen and/or RAID-CRC Symptomatic kit. The files must be filled in with the quantitative information extracted from the software associated with the thermocycler following the manufacturer's instructions. The necessary information is the following:

	A	B	C	D	E
1	Well Name	Dye	Cq (ΔR)		
2	1578	FAM	9,32		
3	1578	HEX	27,65		
4	1578	CY5	33,74		
5	1578	ROX	19,6		
6	C+	FAM	9,32		
7	C+	HEX	27,65		
8	C+	CY5	33,74		
9	C+	ROX	19,6		
10	NTC	FAM	9,32		
11	NTC	HEX	27,65		
12	NTC	CY5	33,74		
13	NTC	ROX	19,6		
14					

The values shown in the image above are example values. You also need to add the values of the qPCR controls indicating the positive control as C+ and the negative control as NTC.

- When completing the required fields, the "Create Analysis" button is enabled. There is a "back" icon, which when pressed cancels the operation and returns to the analysis list.
- Through bulk uploading, multiple analysis files that have been performed with the same qPCR kit (same serial number) can be entered into the system. For the use of this functionality, the files must have a name with a specific format for proper reading. The necessary format is specified on the same screen, along with an example

- *By pressing the "Create Analysis" button, the system processes the selected file. If it is processed correctly, the system displays the following message: "The study has been added successfully".*
 - *If the file is processed incorrectly, the system displays the following message: "The attached file does not contain the correct format".*
- *For RAID-Dx analyses:*
- *The RAID-Dx icon is selected from the main menu or it can be also selected from the side menu.*
 - *The user will see a list of all the RAID-Dx analyses that he has carried out. If this is the first time you log in and no analyses have been made, a message will appear stating that there is no RAID-Dx analysis.*
 - *To carry out a new analysis, the user must ensure it exists an habilitated kit. To proceed with the analysis, the user must press the "New Analysis" button. By pressing the button, the system opens a loading screen, which has the following fields:*
 - *Hospital: Combo to select the hospital or centre in which the analysis is carried out. It is a required field.*
 - *Thermocycler: Combo to select the thermal cycler with which the samples have been analysed. It is a required field.*
 - *Different fields to attach the corresponding files. It only supports files in "*" .xlsx or * .xls" format. It is a required field. For RAID-Dx analysis, 3 different files are required (Dx_GG1, Dx_GG2, and Dx_GG3).*
 - *The system also presents a "Download template" option, in which the user can download a correct template model of each required file so that they can be completed without errors.*

Each template (Dx_GG1, Dx_GG2, and Dx_GG3) has a single tab that must be filled in following the instructions for use of the RAID-Dx kit. The files must be filled in with the quantitative information extracted from the software

associated with the thermocycler following the manufacturer's instructions.
The necessary information is the following:

	A	B	C	D
1	Well Name	Dye	Cq (ΔR)	
2	536	FAM	12,47	
3	536	HEX	14,71	
4	536	ROX	17,1	
5	C+	FAM	12,47	
6	C+	HEX	14,71	
7	C+	ROX	17,1	
8	NTC	FAM	12,47	
9	NTC	HEX	14,71	
10	NTC	ROX	17,1	

The values shown in the image above are example values. You also need to add the values of the qPCR controls indicating the positive control as C+ and the negative control as NTC.

- When completing the required fields, the "Create Analysis" button is enabled. There is a "back" icon, which when pressed cancels the operation and returns to the analysis list.
- Through bulk uploading, multiple analysis files that have been performed with the same qPCR kit (same serial number) can be entered into the system. For the use of this functionality, the files must have a name with a specific format for proper system reading. The necessary format is specified on the same screen, along with an example.
- By pressing the "Create Analysis" button, the system processes the selected file. If it is processed correctly, the system displays the following message: "The study has been added successfully".
- If the file is processed incorrectly, the system displays the following message: "The attached file does not contain the correct format".

- For TestUrGut analyses:

- *The TestUrGut icon is selected from the main menu or it can be also selected from the side menu.*
- *The user will see a list of all the TestUrGut analyses that he has carried out. If this is the first time you log in and no analyses have been made, a message will appear stating that there is no TestUrGut analysis.*
- *To carry out a new analysis, the user must ensure it exists an habilitated kit. To proceed with the analysis, the user must press the "New Analysis" button. By pressing the button, the system opens a loading screen, which has the following fields:*
 - *Hospital: Combo to select the hospital or centre in which the analysis is carried out. It is a required field.*
 - *Thermocycler: Combo to select the thermal cycler with which the samples have been analysed. It is a required field.*
 - *Different fields to attach the corresponding files. It only supports files in ".xlsx or *.xls" format. It is a required field. For TestUrGut analysis, 6 different files are required (TUG_1, TUG_2, TUG_3, TUG_4, TUG_5, and TUG_6).*
 - *The system also presents a "Download template" option, in which the user can download a correct template model of each required file so that they can be completed without errors.*
- *Each template (TUG_1, TUG_2, TUG_3, TUG_4, TUG_5, and TUG_6) has a single tab that must be filled in following the instructions for use of the TestUrGut kit. The files must be filled in with the quantitative information extracted from the software associated with the thermocycler following the manufacturer's instructions. The necessary information is the following:*

	A	B	C	D
1	Well Name	Dye	Cq (ΔR)	
2	536	FAM	12,47	
3	536	HEX	14,71	
4	536	ROX	17,1	
5	C+	FAM	12,47	
6	C+	HEX	14,71	
7	C+	ROX	17,1	
8	NTC	FAM	12,47	
9	NTC	HEX	14,71	
10	NTC	ROX	17,1	

The values shown in the image above are example values. You also need to add the values of the qPCR controls indicating the positive control as C+ and the negative control as NTC.

- When completing the required fields, the “Create Analysis” button is enabled. There is a “back” icon, which when pressed cancels the operation and returns to the analysis list.
- Through bulk uploading, multiple analysis files that have been performed with the same qPCR kit (same serial number) can be entered into the system. For the use of this functionality, the files must have a name with a specific format for proper system reading. The necessary format is specified on the same screen, along with an example
 - By pressing the “Create Analysis” button, the system processes the selected file. If it is processed correctly, the system displays the following message: “The study has been added successfully”.
 - If the file is processed incorrectly, the system displays the following message: “The attached file does not contain the correct format”.

Once the analysis is created, the user has 4 different functions:

- The user has the option of placing observations by pressing the icon: . A pop-up opens with a text field destined to enter, delete, or modify an observation, it will be related to the studies that were analysed at the same time.
 - The user has the option to view, edit, or reset the comment associated with the results by pressing the icon: . A pop-up opens with a text field where comments can be viewed, edited, and reset. This field accepts up to 1000 characters.
 - The user has the option to view and download the report by pressing the icon: . A screen opens with the visualization of the report, which can be downloaded using the "Download report" button to obtain a PDF file with the results report.
 - The user has the option of downloading the excel file that was entered into the system to analyse the study by pressing the icon: 
- Analyses search:
- Upon entering any of the product analysis modules, the system enables a collapsible search option. When displaying this option the system enables the following filters for the search:
 - Date range: a calendar is enabled to select the From and To of the Analysis
 - Well name: alphanumeric field
 - Clinical history: alphanumeric field
 - Hospitals: combo, this option depends on the hospitals where the hospital administrator is associated.
 - Technician: combo, with the option for text entry.
 - Lot number: alphanumeric field
 - Kit serial number: alphanumeric field
 - The "Reset Filters" button is available to eliminate the search data selected in the filters.
 - The "Apply" button applies the data selected in the filters. These will be reflected in the results list. The results grid presents the following fields:

- *Date and time of entry of the sample into the system*
- *Well name: sample ID*
- *Clinical history*
- *Status: status in which is the sample analysis*

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The results obtained through the platform are interpreted and delivered to the patient by a trained doctor or health professional.

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The report preparation is done through the GoodGut-Test™ platform. This applies predetermined algorithms that have been developed specifically for each of the kits complemented by the platform. For more information on the kits, consult the technical documentation of these products: RAID-CRC, RAID-Dx, and TestUrGut.

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The instructions for use detailed below are for the user responsible for the interpretation of the results after obtaining a final diagnosis after analysis of the samples with the RAID-CRC, RAID-Dx, and TestUrGut in vitro diagnostic medical devices.

In order to use the platform, the user will receive an email where the GoodGut team welcomes the user and invites him/her to register through a link. The user must use the link to establish their password. After registering, the user can access the screen for the System Login: <http://goodgut-test.eu/auth/login>.

- *You must enter the existing email and password in the system and then press the “Sign in” button.*
- *The user sees the main menu with the shortcuts to each of the analyses, depending on the permissions granted to their centre.*
- *The user may close his/her session from the system at any time by displaying the user menu and pressing the “Log out” option. The system asks if you want to exit the system. If the user presses Yes, the system takes the user to the Login screen of the system. If the user presses No, the system keeps the user at the current screen.*

Doctor User:

- *To view the reports of the different products, the user must select the type of analysis in the main menu using the RAID-CRC, RAID-Dx, and TestUrGut icons. It can also be selected from the side menu.*

- *The user sees a list of all the analyses of the selected products that have been carried out in their associated centre. If there is still no analysis associated with it, a message will appear detailing to the user that there is no analysis.*
- *Upon entering the Analysis module, the system enables a collapsible search option. When displaying this option the system enables the following filters for the search:*
 - *Date range: a calendar is enabled to select from and until the analysis*
 - *Sample code: alphanumeric field*
 - *Clinical history: alphanumeric field*
 - *Results: combo Positive/Negative regarding RAID-CRC and Positive/Not compatible in the case of RAID-Dx.*
 - *Hospitals: combo, this option depends on the hospitals where the hospital administrator is associated.*
 - *Technician: combo, this option depends on the number of technicians in the selected hospital.*
 - *Route of entry: combo Screening/Symptomatic in the case of RAID-CRC.*
 - *Status: combo, this option refers to the state of the analysis associated with the sample.*
 - *Kit series number: alphanumeric field*
 - *Lot number: alphanumeric field.*
- *The “Reset Filters” button is available to eliminate the search data selected in the filters.*
- *The “Apply” button applies the data selected in the filters. These will be reflected in the results list. The results grid presents the following fields:*
 - *Date and time: date of analysis entrance to the system*
 - *Sample code: it is obtained from the files that are entered in the system*
 - *Clinical history: it is obtained from the file that is entered into the system*
 - *Route of entry: refers to the Screening / Symptomatic analysis type in the case of RAID-CRC.*
 - *Results: Positive/Negative in the case of RAID-CRC and Positive/Not compatible in the case of RAID-Dx.*

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- *The user has the option of placing observations by pressing the icon:  . A pop-up opens with a text field destined to enter, delete, or modify an observation, it will be related to the studies that were analysed at the same time.*
- *The user has the option to view, edit, or reset the comment associated with the results by pressing the icon:  . A pop-up opens with a text field where comments can be viewed, edited, and reset. This field accepts up to 1000 characters.*
- *The user has the option to view and download the report by pressing the icon:  . A screen opens with the visualization of the report, which can be downloaded using the "Download report" button to obtain a PDF file with the results report.*
- *The user has the option of downloading the excel file that was entered into the system to analyse the study by pressing the icon: *

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More details of the predesigned algorithms and their development can be found in the technical documents of the different products supplemented by the platform.

- *RAID-CRC technical document*
- *RAID-Dx technical document*
- *TestUrGut technical document*

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There have been four significant changes since the release of the platform (CC-21003, CC-22003, CC-23004, and CC-23005).

Change CC-21003 has improved the system without presenting a significant impact on the interface of the hospital administrator user, technical user, or medical user. The change presents a minimal impact on the usability of the technical user.

The RAID-Dx product has been improved and now its analysis in the laboratory consists of 3 multiplexes. With this modification, the obtaining of the results from the thermal cycler has been modified, so the template that must be attached for the analysis of the RAID-Dx test has also been modified. The change is minimal and favours the technical user. The template must be filled in following the instructions for the use of the RAID-Dx kit.

The CC-22003 change has improved the system usability, showing a significant impact on the interface of the technical user but not the hospital administrator or medical user.

- *The change has allowed the technical user to enter the samples and information associated with the samples upon receipt and then add the sample analysis results, whereas previously the information had to be uploaded to the platform at the same time.*
- *The number of files to upload for each product has also been modified so that human interaction in the analysis of raw data is minimal.*
- *Besides, the platform detects the control values of the different qPCRs during the analysis of the samples, notifying the technical user in case the analyses must be repeated.*

- *A qPCR kit checkpoint the user uses to analyse samples with a warning when the kit is expired has been added.*

The CC-23004 change has enhanced the system's usability, presenting a significant impact on the technical user interface. The change has a minimal effect on the usability of hospital administrators or medical users.

- *The change enables technical users to automatically generate a results report with a negative diagnosis when inputting RAID-CRC Screen and Symptomatic samples with a FIT value below the established cutoff point for analysis.*
- *In addition, for the creation of a TesturGut analysis, it is no longer necessary to input the sample weight.*
- *Finally, an indicator (arrow) has also been added next to the traffic light symbol in the TestUrGut results reports to signify whether the abundance of markers is above or below the optimal tolerance range, thereby facilitating result comprehension.*

The CC-23005 change has improved the system's usability, presenting a significant impact on the technical user interface. The change has minimal impact on usability for hospital administrators or medical users.

- *The change has enabled both the automation of loading multiple analyses and the inclusion of comments in the results report for all products.*
- *In line with the CC-23002 change and following established guidelines, the control associated with the established threshold for analysis has been removed from the system.*
- *The requirement to enter UDI information during analysis loading has also been removed from the system. In line with this change, the Lots/Kits menu has been modified to establish semi-automatic control of usage.*
- *To facilitate the interpretation of TestUrGut results reports, traffic lights, and indicator configurations have been improved.*
- *Finally, all system functionalities have been translated into Italian to enhance accessibility and usability.*